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EN
Instruction for use
Rigid gas permeable contact lens - orthokeratology
Name of the product: see packing label.
Name of the material: see packing label.
Storage solution: (if present) poloxamer 0,25%, EDTA 0,10% and polyhexanide 0,0002%, in a buffered isotonic and sterile solution.
Intended use: correction of myopia by temporary controlled reshaping of the cornea (orthokeratology) and reduction of the progression of myopia (myopia management) for users of 12 years and older.
Wearing schedule: the lenses are suitable for orthokeratology. Follow the wear and replacement schedule provided by your eye care professional.

IT
Istruzioni for cleaning and use
Insering the lens: (Fig. 1) Removing the lens: (Fig. 2) Centering the lens: (Fig. 3)
<ul style="list-style-type: none">The lenses and the contact lens holder should be regularly cleaned and looked after in accordance with the instructions provided by your eye care professional. Do not use tap water to rinse the lenses. Saline (sterile salt solution) is preferred and allow your lenses case to air dry while open. Use only the care system recommended by your eye care professional to avoid unwanted interaction between the contact lenses and the care system. If lenses are not being used for a longer period: Clean the contact lenses thoroughly. Clean the lenses dry in a clean and absolutely dry contact lens holder. Clean the lenses immediately before you use them again. Keep the lenses for 8 hours in a stored solution for a better comfort. Wash your hands thoroughly with water and soap and dry them carefully before handling lenses.Do not use in case the contact lenses is damaged. Please dispose of contact lenses via normal household waste unless local regulations require otherwise.
Warning
Do not use and consult your eye care professional if any of the following conditions occur: <ul style="list-style-type: none">Inflammation, infection, redness or irritation of the eye or eyelid. Change in cornea sensitivity or reduced tear production. Any general disease that may affect the eye, or for which wearing contact lenses may increase the risk of complications. Intolerance or abnormal eye reaction to the wearing of contact lenses.
Report any changes in eye comfort while wearing the contact lenses to your eye care professional; these changes may include increased tearing, severe redness, poor vision or allergic reaction. Failing to seek timely medical treatment may result in permanent eye damage.
Do not use contact lenses if the contact lens wearer cannot or refuses to follow the instructions for safe contact lens use.

Parental guidance is required for minors and parents/guardians should be informed by the eye care professional concerning risks and benefits of wearing contact lenses.
Do not expose your contact lenses to any water: tap, bottled, distilled, lake or open body water. Never use non-sterile water (distilled water, tap water or any homemade saline solution). Tap and distilled water have been associated with Acanthamoeba keratitis, a corneal infection that is resistant to treatment and cure. Remove your contact lenses before swimming. There is a risk of eye infection from bacteria in swimming pool water, hot tubs, lakes and

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